

PIG Surgical Floor Skimmer

Collects and extracts fluid like blood and saline for safe and dry O.R. floors.

Features / Benefits:

- **Shaped for efficiency** — captures liquids from all angles during high fluid surgeries for fast, easy cleanup
- **High-absorbency wicking pad** — proprietary bottom wicking pad removes 60 liters per hour of blood, saline and other fluids
- **Easy-slide floor pad** — easily reposition with your foot; keeps arms and hands free
- **Safe, Dry Floors** — helps maintain clean and dry O.R. floors to prevent risk of slips, trips and falls
- **Low-profile design** — it's so thin, the surgeon will never know it's there
- **Silent suction** — helps keep the operating room quiet
- **12-foot suction tubing** — long tubing length allows access to all areas around the O.R. table (included with HC200)



9" x 9" Footprint



Bottom Wicking Pad

PIG™ Surgical Floor Skimmer

Item #	Description
HC200	10 Skimmers w/tubing
HC201	20 Skimmers

NOTE: PIG Surgical Floor Skimmers are designed for one-time use. Please dispose of properly.


Flip over for product details >




New Pig

HOW TO USE


PIG Surgical Floor Skimmer

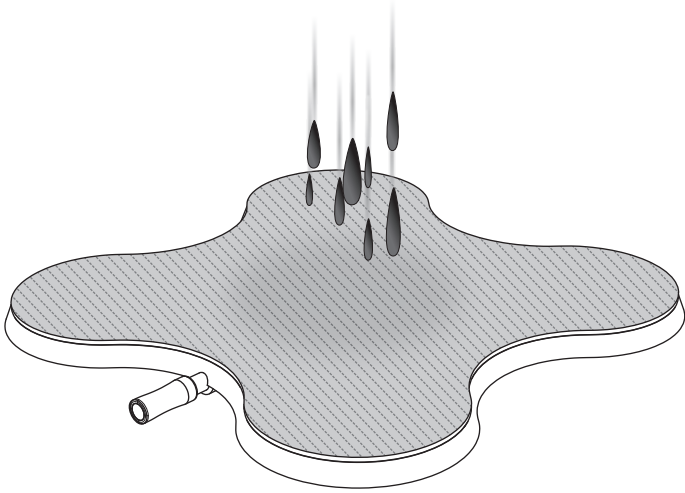


1. Wet entire surface of bottom pad before use.

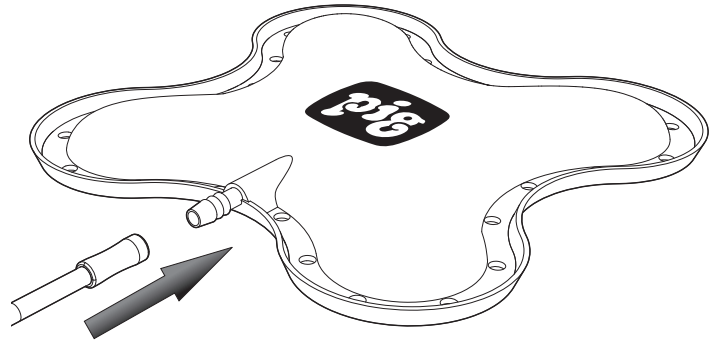

2. Securely attach either end of suction tubing to barb fitting.

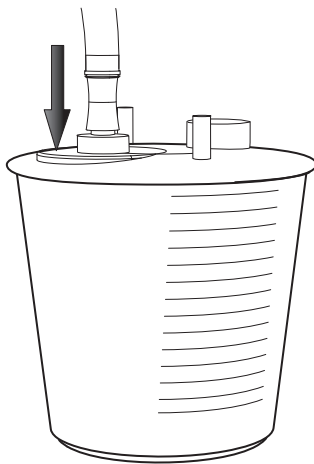

3. Securely connect the other end of tubing to your fluid collection canister. Turn on your vacuum source when ready.

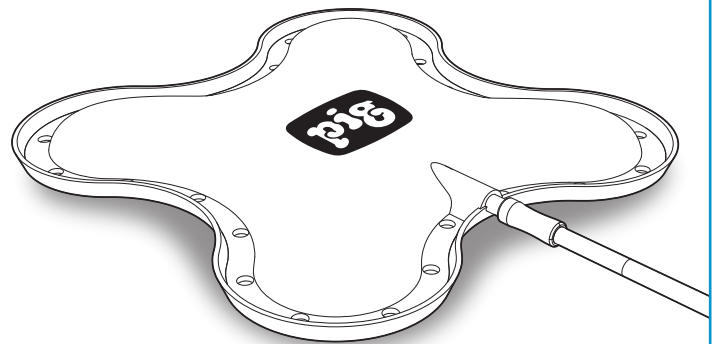

4. Place skimmer wherever fluids are puddling — before, during or after a procedure.

NOTE: PIG Surgical Floor Skimmers are designed for one-time use. Please dispose of properly.



New Pig

[Flip over for product details >](#)