



# PIG Surgical Absorbent Mat Roll Dispenser


## Instructions

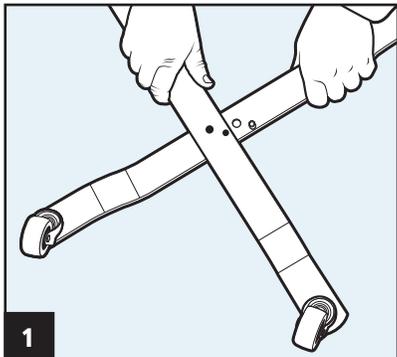


**Step 1** – Cross bases to form an “X” with wheels facing up.

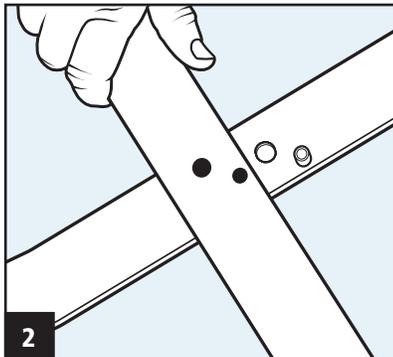


**Step 2** – Align pin with pinhole to hold base in position.

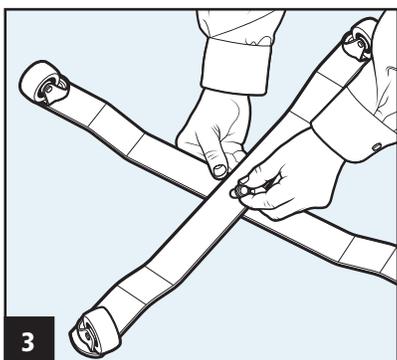


**Step 3** – Insert bolt through both bases. Threads should be pointing up when base is upright.

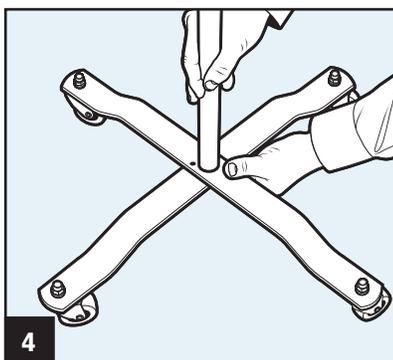

**Step 4** – Screw either end of pole onto threaded end of bolt.

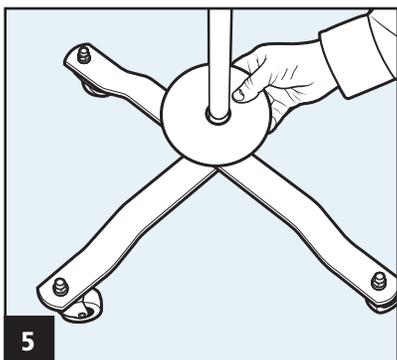


**Step 5** – Slide friction disc over pole so it rests on the base.

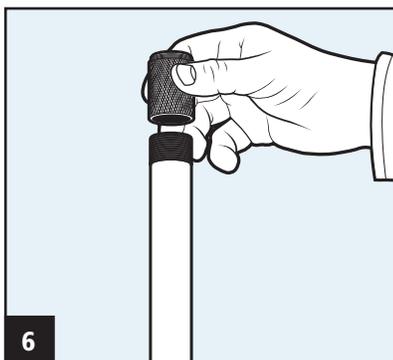

**Step 6** – Screw connector onto top of pole for a finished look.

Clip keeps mat from unfurling. Magnet sticks to pole when not in use.

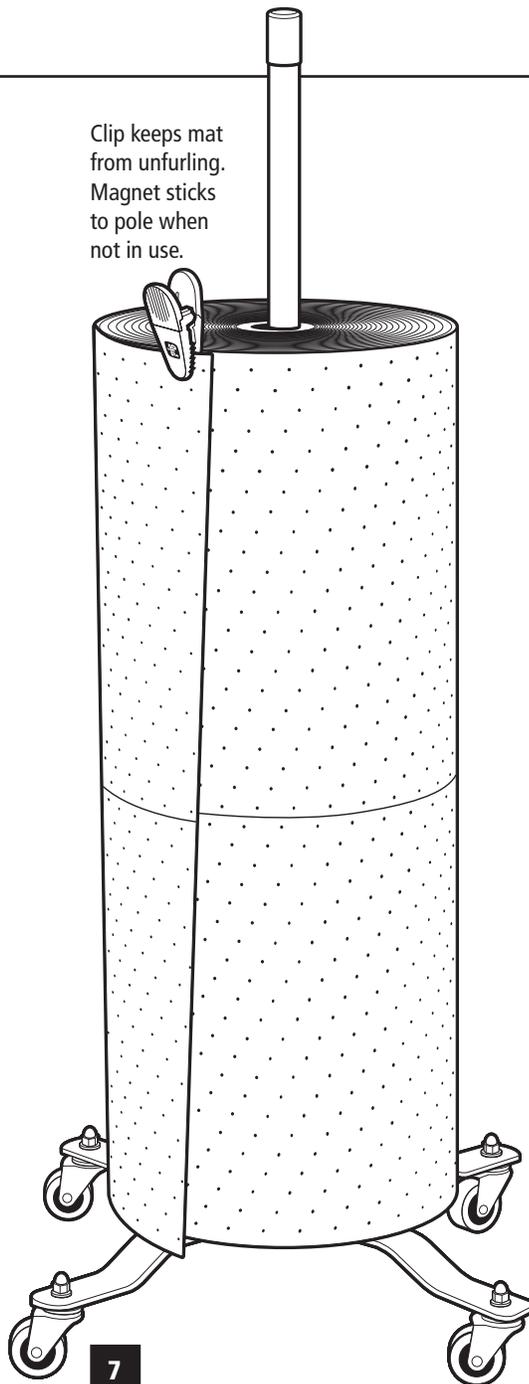

**Step 7** – Load mat onto pole so it rests on the friction disc. Your PIG Mat is ready to roll!



**New Pig**

One Pork Avenue  
Tipton, PA 16684-0304

[newpig.com](http://newpig.com)

North America: 1-800-468-4647

Europe: +31 (0)76 596 92 50

China: +86-21-400 921 5178

PIG, PIG logo are registered trademarks in USA and other countries. See [tm.newpig.com](http://tm.newpig.com)

UK: 0800 919 900

Outside North America: +1-814-684-0101

166026 / 354066