

PRODUCT DETAILS

PIG Surgical Absorbent Mat with Adhesive Backing

Absorbs saline and blood and stays put to keep O.R. floors dry and safe during high fluid surgeries.

Features/Benefits:

- **Adhesive backing** — position mat on dry floor before procedures and it stays put; no sliding, shifting or bunching up
- **Highly absorbent** — absorbs up to 3 liters of blood, urine, vomit, saline and other O.R. fluids to help keep your floor clean and safe
- **Large footprint** — 31" x 40" mat covers large area of O.R. floor
- **Durable** — mat pads won't rip, tear or fray under light foot traffic or mayo stands and other carts
- **Hygienic** — disposable mats keep surgical environment clean; helps reduce cross-contamination
- **Leakproof** — liquids will not soak through to the floor when saturated
- **Quick and efficient** — mat pad goes down easily and comes up quickly; speeds up O.R. turnover time
- **Use anywhere fluids collect in healthcare facilities** — surgical centers, veterinary clinics, operating rooms, scrub sinks, lab stations and more

NOTE: PIG Surgical Absorbent Mats with Adhesive Backing are designed for one-time use. Please dispose of properly.

PIG® Surgical Absorbent Mat with Adhesive Backing

Item #	Description
HC102	20 Mats • 31" x 40"
HC101	1 Roll • 31" x 100'
HC300	Roll Dispenser



HC101

Available in rolls or mats.

HC102



[Flip over for instructions >](#)



New Pig

HOW TO USE


PIG Surgical Absorbent Mat with Adhesive Backing

PIG Surgical Absorbent Mat with Adhesive Backing should only be applied to your floor before each procedure and left in place for the duration of the surgery. (If additional absorbents are required during a procedure, use our standard PIG Surgical Absorbent Mat.)

1. Make sure the surface you want to cover is clean, dry and free of debris.
2. Place mat on floor with adhesive backing side down.
3. Press down along the edges and into the center of the mat until it is smooth, flat and securely in place, with no air pockets or creases.
4. When the procedure is complete, just pick up the mat and throw it away.

WARNING: Used absorbents can take on the hazardous characteristics of liquids absorbed. Follow local, state and federal regulations when storing or disposing of used absorbents.

NOTE: PIG Surgical Absorbent Mats with Adhesive Backing are designed for one-time use. Please dispose of properly.

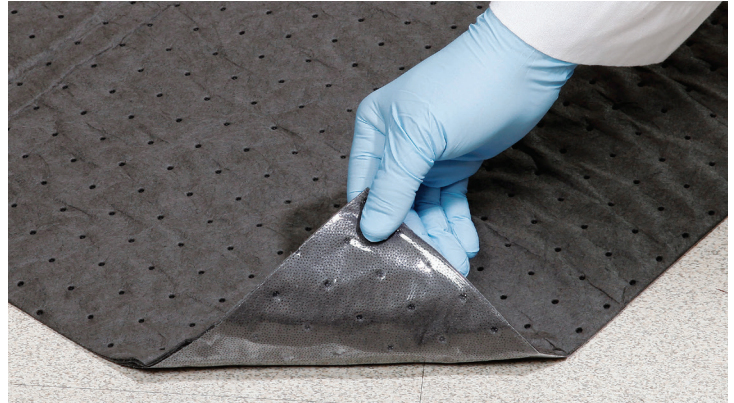

Place with adhesive backing side down to eliminate bunching, curling or sliding during procedures.



Mat stays put to eliminate tripping while absorbing O.R. fluids.

[Flip over for product details >](#)



New Pig