PIG® Hazardous and Chemotherapy Drug Spill Kit – Low Volume

This report is offered as a guide and was developed from information which, to the best of New Pig's knowledge, was reliable and accurate. Due to variables and conditions of application beyond New Pig's control, none of the data shown in this guide is to be construed as a guarantee, expressed or implied. New Pig assumes no responsibility, obligation, or liability in conjunction with the use or misuse of the information.

New Pig

One Pork Avenue
Tipton, PA 16684-0304

newpig.com

North America: **1-800-468-4647**

Europe: **+31 (0)76 596 92 50**

China: **+86-21-400 921 5178**

UK: **0800 919 900**

Outside North America: **+1-814-684-0101**

PIG, PIG logo are registered trademarks in USA and other countries. See tm.newpig.com

166173 / 366836

PIG® Hazardous and Chemotherapy Drug Spill Kit – Low Volume

Includes:

- 2 - PIG® High-Visibility Absorbent Mat Pads
- 1 - Sterile Chemo Gown
- 1 - Pair of Size Large Boot Covers
- 1 - Bouffant Cap
- 2 - Pairs of Size Large Nitrile Gloves
- 1 - Face Shield
- 1 - Hazardous Material Disposal Bag

Instructions:

Step 1 — Wear appropriate PPE

Put on appropriate personal protective equipment (PPE) prior to entering the spill area. (fig. A)

Step 2 — Absorb the liquid

Working from the outside of the spill inward, place mats on the spill until the liquid is completely covered and absorbed. (fig. B)

Step 3 — Dispose of used absorbents

Place all used absorbents or other contaminated materials in the disposal bags and tie the bag closed. Dispose of the bag and all used materials in compliance with local, state and federal regulations. (fig. C)

WARNING: To ensure safety when using enclosed PPE, please consult chemical compatibility guide(s) to confirm that enclosed PPE is compatible with your particular chemicals and fluids; or confirm compatibility with the chemical manufacturer.

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