

Ansell Healthcare Products LLC MICROTOUCH[®] Nitrile E.P.[®] Gloves

The guide on the following page(s) was provided by the supplier.
New Pig Corporation assumes no responsibility, obligation, or
liability in conjunction with the use or misuse of the information.



New Pig

One Pork Avenue
Tipton, PA 16684-0304

newpig.com

North America: **1-800-468-4647**

Europe: **+31 (0)76 596 92 50**

China: **+86-21-400 921 5178**

UK: **0800 919 900**

Outside North America: **+1-814-684-0101**

PIG, PIG logo are registered trademarks in USA and other countries. See tm.newpig.com

358760

March 28, 2019

Gloves used for protection against chemotherapy drug exposure must be selected specifically for the type of drugs being used. Tested for use with Chemotherapy Drugs using ASTM D6978 with average normalized breakthrough times in minutes. The list in Table 1 has been cleared by FDA.

Chemotherapy Drug	Micro-Touch Nitrile E.P.
5-Fluorouracil (Adrucil)	> 240
Carmustine	71.3
Cisplatin	> 240
Cyclophosphamide (Cytoxan)	> 240
Dacarbazine	> 240
Doxorubicin Hydrochloride (Adriamycin)	> 240
Etoposide	> 240
Paclitaxel (Taxol)	> 240
Thio-TEPA	179.5

Additional testing was conducted by an accredited third party laboratory and is presented in the table below.

Chemotherapy Drug	Micro-Touch Nitrile E.P.
Blenoxane	> 240
Busulfan	> 240
Docetaxel	> 240
Fludarabine	> 240
Gemcitabine	> 240
Irinotecan	> 240
Methotrexate	> 240
Mitomycin	> 240
Mitoxantrone	> 240
Oxaliplatin	> 240
Vincristine sulfate	> 240

Sincerely,



Jennifer Singh
Associate Director, Scientific Affairs
Jennifer.Singh@ansell.com